

# SURE CHECK<sup>®</sup> HIV 1/2 Assay



*One Drop of Blood is All You Need*

An FDA approved, rapid point-of-care test that detects antibodies to HIV-1 and HIV-2 in fingerstick whole blood, venous whole blood, serum and plasma specimens.

## Innovative

- Unique Barrel Design: **All-in-one** sample collection and test device
- World's smallest sample size: **2.5 µl**

## Convenient

- A self-contained unit with minimal hands-on-time
- CLIA waived for fingerstick and venous whole blood
- Results in 15 minutes

## Reliable

- Sensitivity: **99.7%**; Specificity: **99.9%\***
- Built-in procedural control

\* Refer to product insert for details

# Test for HIV in 3 Easy Steps with SURE CHECK®

Please refer to the complete product insert provided in the test kit

## 1 Prepare

Fill the testing device with 2.5 µl fingerstick blood



## 2 Run

Push firmly through the foil cap until 3 snaps are felt



## 3 Read

WAIT  
**15**  
MINUTES



NON-REACTIVE

REACTIVE

INVALID

## Product Information

Information Type	Product Detail
Method	Lateral Flow
Time to Results	15 minutes
Test and External Controls Shelf Life**	24 months
Storage Conditions	8 - 30°C (46-86°F)

## Ordering Information

Product	Catalog Number
SURE CHECK® HIV 1/2 Assay (25 Tests) FDA Approved, CLIA Waived	60-9507-0
HIV Rapid Test Control Pack	60-9549-0

\*\* From date of manufacture

Reimbursement Codes: 86703 or 86703QW  
G0435 (for Medicare)

## For More Information

Visit: [www.chembio.com](http://www.chembio.com)  
Call: 1.844.CHEMBIO  
Email: [info@chembio.com](mailto:info@chembio.com)