

DRUG PARENT CHART

PANEL STRIPS

Blue bars below represent test panels strips built into Identify Diagnostics USA cup and dip tests. Your test may include one or more panels based on your specific configuration.

BRAND[®] / STREET NAMES

Green tabs below are common drug brands or street names. Not all drug brands or street names are included in this list. The list of drug compounds under each blue bar indicate what drugs are detectable by that panel and at what cut-off level in ng/mL.

DRUG COMPOUND	Cut-off in ng/ml	DRUG COMPOUND	Cut-off in ng/ml
AMPHETAMINE 500 AMP		MARIJUANA 50 THC	
D-Amphetamine ADDERALL[®]	500	11-nor- Δ^9 -THC-9-COOH POT, WEED	50
L-Amphetamine	50,000	(+/-)-11-Hydroxy- Δ^9 -THC	5,000
Phentermine	45,000	(-)- Δ^8 -THC	20,000
MDA	8,000	(-)- Δ^9 -THC	20,000
BARBITURATES 300 BAR		METHADONE 300 MTD	
Secobarbital SECONAL[®]	300	Methadone DOLOPHINE[®] METHADOSE[®]	300
Phenobarbital LUMINAL[®]	300		
Butalbital FORTABS[®]	300	METHAMPHETAMINES 500 MET	
Pentobarbital NEMBUTAL[®]	250	D-Methamphetamine DESOXYN[®]	500
Amobarbital AMYTAL[®]	2,500	D-Methamphetamine	50,000
Cyclopentobarbital	500	L-Amphetamine	50,000
Aprobarbital	500	1R,2S(-)-Ephedrine	100,000
Butabarbital BUTISOL[®]	100	MDMA	3,500
BENZODIAZEPINES 300 BZO		METHYLENEDIOXYMETHAMPHETAMINE 500 MDMA	
Oxazepam SERAX[®]	300	(+/-)-MDMA ECSTASY	500
Alprazolam XANAX[®]	200	(+/-)-MDA	3,900
a-Hydroxyalprazolam	1,900	(+/-)-MDEA	500
Bromazepam LECTOPAM[®] LEXOTANIL[®]	1,000		
Clobazam ONFI[®]	200	OPIATE 300 OPI300	
Clorazepate TRANXENE[®]	750	Morphine HEROIN METABOLITE ROXANOL[®]	300
Diazepam VALIUM[®]	1,000	Codeine TYLENOL 3[®]	100
Flunitrazepam	250	Hydrocodone LORATAB[®] LORCET[®] VICODIN[®]	1,250
Lorazepam ATIVAN[®]	3,900	Hydromorphone DILAUDID[®] OPIDOL[®]	2,500
Lorazepam-Glucuronide	5,000	Levorphanol	50,000
Nitrazepam	250	Morphine 3-Glucuronide	400
Norchlordiazepoxide	500	Norcodeine	6,000
Nordazepam	390	Oxycodone OXYCONTIN[®] PERCOCET[®]	75,000
Nordiazepoxide	400	Thebaine	90,000
Temazepam RESTORIL[®]	150		
Triazolam HALCION[®]	2,500	OXYCODONE 100 OXY	
		Oxycodone OXYCONTIN[®] PERCOCET[®]	100
BUPRENORPHINE 10 BUP		Codeine TYLENOL 3[®]	50,000
Buprenorphine SUBOXONE[®] SUBUTEX[®]	10	Ethylmorphine	50,000
		Hydrocodone LORATAB[®] LORCET[®] VICODIN[®]	5,000
COCAINE 150 COC		Hydromorphone DILAUDID[®] OPIDOL[®]	25,000
Cocaine COKE, CRACK	5,000	Oxymorphone OPANA[®]	12,500
Benzoylcegonine	150		
Ecgonine	50,000	PHENCYCLIDINE 25 PCP	
Cocaethylene	50,000	Phencyclidine ANGEL DUST	25
COTININE 200 COT		TRAMADOL 100 TRA	
Cotinine NICOTINE	200	Tramadol CONZIP[®] ULTRACET[®] ULTRAM[®]	100
EDDP 300 EDDP		SYNTHETIC MARIJUANA 25 K2	
EDDP Methadone Metabolite	300	Synthetic Marijuana SPICE	25
ETHYL CLUCURONIDE 500 ETG		TRICYCLIC ANTIDEPRESSANTS 1000 TCA	
Ethyl Glucuronide ALCOHOL	500	Notriptiline PAMELOR[®]	1,000
		Amitriptyline ELAVIL[®]	4,000
FENTANYL 25 FEN		Clomipramine ANAFRANIL[®]	2,000
Fentanyl DURAGESIC[®] ABSTRAL[®] LONYSYS[®]	25	Desipramine NORPRAMIN[®]	500
		Doxepin SILENOR[®] ZONALON[®]	1,000
GABAPENTIN 2000 GAB		Imipramine TOFRANIL[®]	1,000
Gabapentin GRALISE[®] NEURONTIN[®]	2,000	Promethazine	1,000
		Trimipramine SURMONTIL[®]	5,000
KRATOM 100 KRA			
Kratom	100		