

# PROCEDURE CARD


## Multi-Drug Oral Fluid Test



Scan for how-to training information including a certification quiz.





Scan for printable drug test result form PDF files.

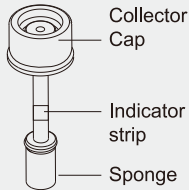


### STEP 1

- Remove the collector and cup from the sealed pouch.

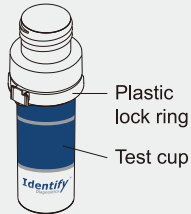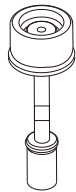


### STEP 2

- Insert the sponge end into the mouth, actively swab around the gums on both sides of the mouth and place the sponge end under the tongue for a total of 2-3 minutes until the sponge becomes fully saturated and indicator strip turns red.

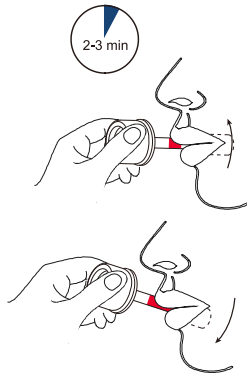


### STEP 3

- Place saturated oral fluid collector into test cup and screw the collector to press sponge fully to release oral fluid. Place the test cup on a clean and level surface and then leave it for 3-5 minutes.

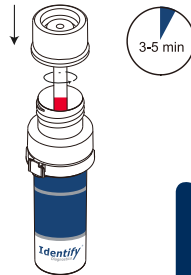

### STEP 4

- Remove the plastic lock ring and screw the lid and start a timer.

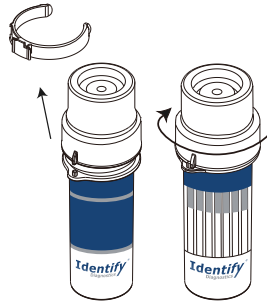


### STEP 5

- Read the test results at 3-5 minutes.

If all lines are clearly visible at 3 minutes or sooner, then the test can be interpreted as negative and discarded. If the sample does not migrate in the test cup even after 3 minutes, please slide the device from side to side. If any lines are not visible at 3 minutes, then the test should be re-read at 5 minutes.

### Result Interpretation

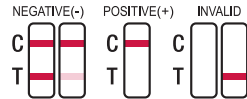

For Alcohol strip, when applicable, the results should be read at 2 minutes. Compare the color of the reaction pad with the chart provided separately/on foil pouch to determine the relative oral fluid alcohol level.

### Why Identify?

Because you need to know.

[IdentifyDiagnostics.com](http://IdentifyDiagnostics.com)