



BioSign® Flu A + B

An in vitro rapid qualitative test that detects influenza type A and type B antigens directly from nasal swab, nasopharyngeal swab and nasal aspirate/wash specimens*

**CLIA
WAIVED**

FOR SWAB
SPECIMENS

*Differentiates between
Influenza A and B*


INCLUDES:



**POSITIVE AND
NEGATIVE CONTROL
SWABS**



**INNOVATIVE FLIP
DESIGN WITH
ONBOARD SAMPLE
EXTRACTION**

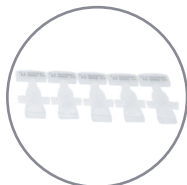


**PRE-MEASURED
DEVELOPER SOLUTION
CAPSULE for increased
accuracy and ease of use**



**FLOCKED NASAL SWABS
for improved patient
comfort and superior
specimen collection**

SIMPLE PROCEDURE

¹ FOR SWAB SPECIMENS

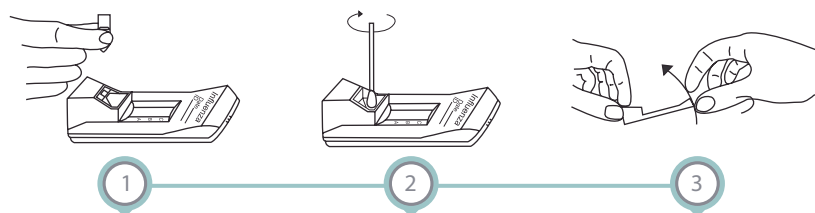

READ RESULTS IN 10 TO 15 MINUTES.

READING RESULTS

VALID TEST RESULTS				INVALID TEST RESULTS		
NEGATIVE	POSITIVE	POSITIVE	POSITIVE	INVALID	INVALID	INVALID
	A LINE: Influenza type A	B LINE: Influenza type A	A & B LINE: Influenza type B	Repeat with new sample and device.		

PROVEN CLINICAL PERFORMANCE

(BioSign® Flu A & B vs, PCR)

	INFLUENZA A	INFLUENZA B
POSITIVE % AGREEMENT	89.2% (95% CI:83.0-93.4%)	86.4% (95% CI:80.1-90.9%)
NEGATIVE % AGREEMENT	99.4% (95% CI:97.7-99.8%)	99.0% (95% CI:97.1-97.7%)

BioSign® Flu A + B, 25 tests/kit, REF #BSP-510-25

CPT CODE: Influenza A 87804QWReimbursement\$1 6.55 ²
Influenza B 87804QW-59.....Reimbursement\$1 6.55 ²

DISTRIBUTED BY:

THE TESTS YOU WANT. THE ACCURACY YOU NEED!



Medical Distribution Group, Inc.

For questions please call: 727-744-2967

2361 Whitfield Park Avenue. Sarasota, FL 34243

¹ See Package Insert for complete details.

² Check with each state for exact reimbursement.

* Nasal Aspirate Specimens - Not CLIA WAIVED.

* CPT reimbursements vary, check your state for current amounts