



Identify Health Drug Test Dip Training Quiz

1. The Identify Health Drug Test Dip Card test is...?

- ☐ A preliminary drug screen
- ☐ A confirmatory test
- ☐ Testing how intoxicated someone is
- ☐ None of the Above

2. Positive results should be read at...?

- ☐ 2 minutes
- ☐ 5 minutes
- ☐ As soon as they appear
- ☐ 60 minutes

3. Which describes a step in preparing a restroom for a urine collection?

- ☐ Turn off water sources (sink faucets)
- ☐ Ensuring toilet and tank water are blue
- ☐ Remove all substances that may be used to adulterate urine specimen (soaps, disinfectants, cleansing agents)
- ☐ All of the above

4. A normal temperature range for urine specimen is...?


- ☐ 80 - 90 degrees Fahrenheit
- ☐ 90 - 100 degrees Fahrenheit
- ☐ 101 - 104 degrees Fahrenheit
- ☐ Not necessary to verify temperature

5. A Identify Health Dip Card may be used after the expiration date if...?

- ☐ The Identify Health Dip Card has remained unopened in its sealed pouch
- ☐ The Identify Health Dip Card has been frozen
- ☐ The Identify Health Dip Card should never be used after expiration date
- ☐ The Identify Health Dip Card is within 1 month of expiration date


6. A test line that is very light in intensity compared to other test lines..?

- ☐ Should be considered invalid
- ☐ Should be interpreted as negative
- ☐ Should be interpreted as a marginal positive
- ☐ Should be re-run since the results are questionable

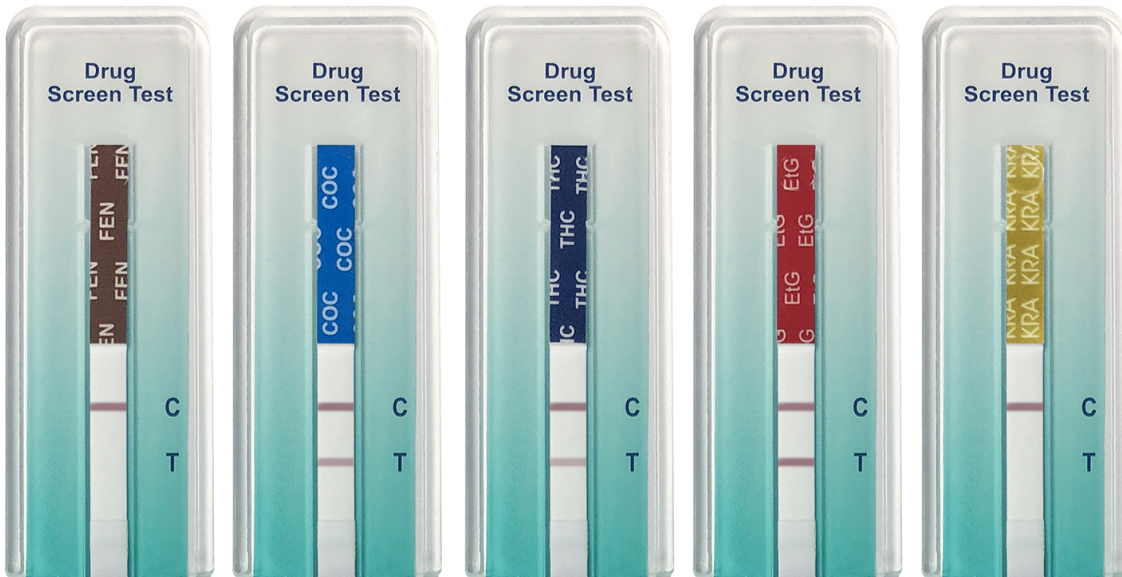


7. Which drug test panel(s) indicates a negative result in the above image?

- ☐ FEN
- ☐ COC
- ☐ THC
- ☐ ETG
- ☐ COC, THC, and ETG

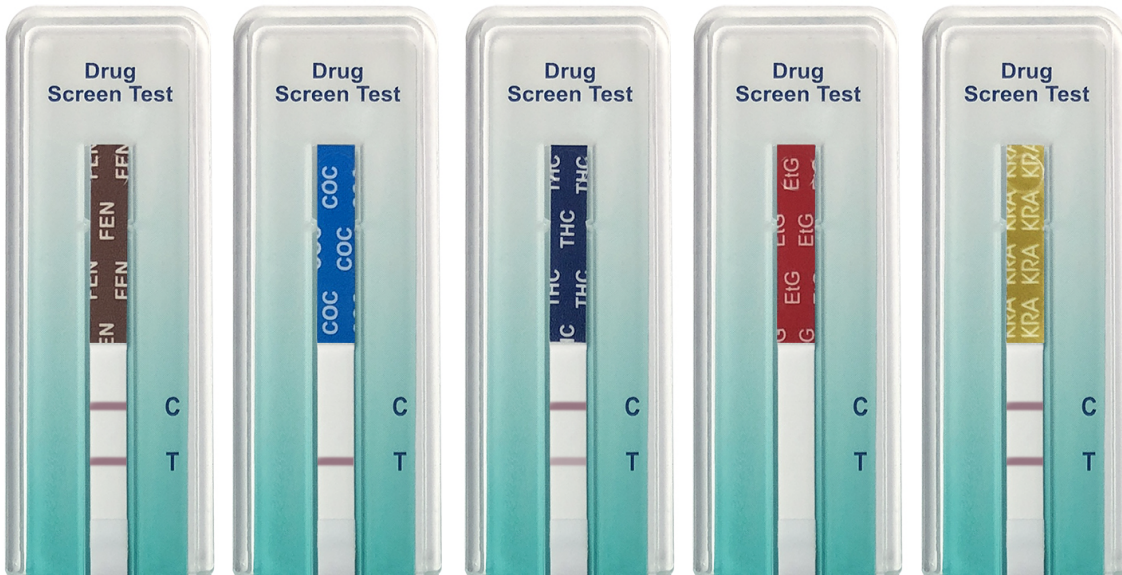

8. Which drug test panel(s) indicates an invalid result in the above image?

- ☐ FEN
- ☐ COC
- ☐ THC
- ☐ ETG
- ☐ COC and ETG

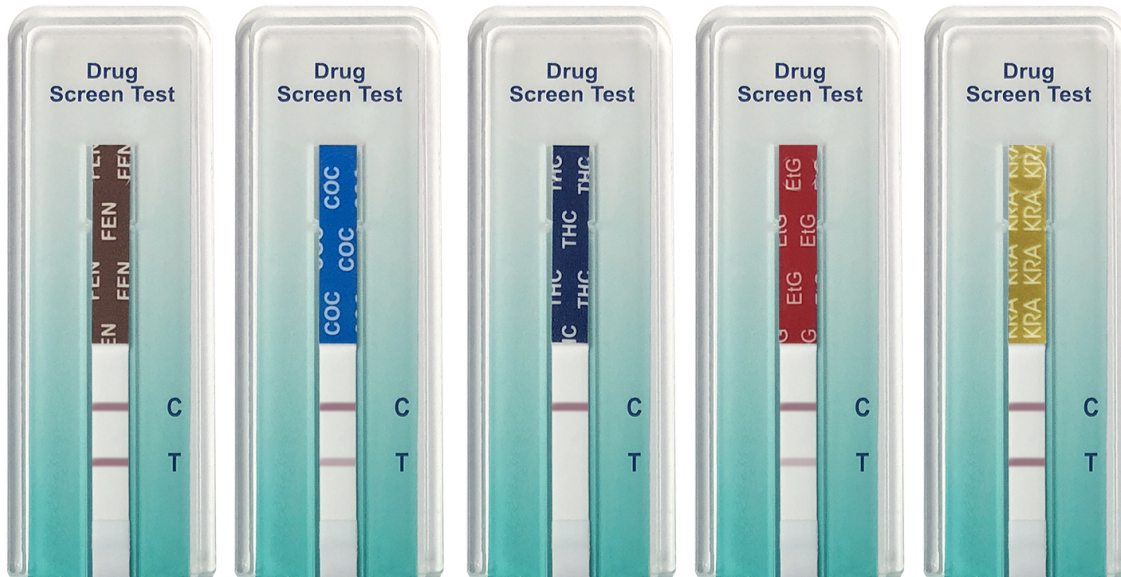

9. Which drug test panel(s) indicates a positive result in the above image?

- ☐ COC
- ☐ THC
- ☐ ETG
- ☐ FEN and KRA

10. The cut-off level of a drug is...?

- ☐ Indicated by the amount of lines that appear per strip
- ☐ The level of urine required to activate the test
- ☐ The concentration level established as a break point or threshold for labeling a result positive or negative
- ☐ 50

11. The test results are stable...?

- ☐ From noon and midnight
- ☐ Between 5-8 minutes
- ☐ For 60 minutes
- ☐ For two hours

12. If a specimen temperature is not within normal range...?

- ☐ Assume the donor recently drank a cold beverage
- ☐ Specimen is invalid
- ☐ If the specimen is negative, it is acceptable
- ☐ If the specimen is positive, it is acceptable

13. An Adulteration Strip should be read...?

- ☐ At the test read time by comparing the colors to the included color chart
- ☐ After 60 minutes
- ☐ By the donor
- ☐ When the control line appears

14. When no control line appears at 5 minutes you should...?

- ☐ Report the result as positive
- ☐ Shake the dip card test
- ☐ Report the result as invalid and repeat the test using a different Identify Health Dip Card with the same sample
- ☐ Report the result as adulterated